### Product And Manufacturer Information

<b>Contents:</b>	1 - Sodium Electrode 1 - O-ring
<b>Intended Use:</b>	Use with Cobas ISE Modules.
<b>Storage:</b>	Store the Sodium electrode at room temperature (18-25°C) until you are ready to use it in your system.
<b>Precautions:</b>	Please check that the electrode has Fill Solution and no air bubbles in the fill chamber near the membrane sample path area prior to installation.

Diamond Diagnostics  
333 Fiske Street, Holliston, MA.

**For Technical Assistance call:**  
Diamond Diagnostics Technical Services at 1-508-429-0450

### Installation

- 1 Remove the green tape.
- 2 Check that the electrode has Fill Solution and the inner electrode touches or is immersed in the fill solution.
- 3 Check for air bubbles in the fill chamber near sample path. If there are any, tap the electrode body to dislodge air bubbles.
- 4 Verify that the O-ring is in place on the electrode body.
- 5 Install electrode in your system.
- 6 Allow the electrode to equilibrate for at least 15 minutes.
- 7 Perform a calibration.
- 8 Analyze a quality control sample.

### Warranty

Warranty is calculated 6 months from the date of shipment from Holliston, MA.

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