**Manufacturer and Product Information**

Diamond Diagnostics, 333 Fiske Street, Holliston, MA.

**For Technical Assistance call:**

Diamond Diagnostics Technical Services at 1-508-429-0450

<b>Intended Use:</b>	To verify the performance of the glucose and lactate sensors on 800 series systems. Each ampule contains enough solution to perform one analysis.
<b>Summary And Principle:</b>	This product is intended to serve as a functional equivalent to pre-existing material distributed by the Original Equipment Manufacturer (OEM).
<b>Reagents:</b>	High G/L, CD-570097D, 10 x 2 mL
<b>Containing:</b>	High G/L is an aqueous solution containing 950 mg/dL (52.72 mmol/L) glucose, 25 mmol/L (240 mg/dL) lactate, salts, buffer, surfactant and preservative.
<b>For <i>in vitro</i> diagnostic use only</b>	
<b>Cautions:</b>	Exercise normal laboratory precautions. If contact occurs with skin, rinse affected area with water. If contact with eyes occurs, immediately rinse with copious amount of clean water or eye rinse. In cases of accidental ingestion, contact a physician immediately.
<b>Stability:</b>	Store High G/L refrigerated at 2-8°C. Package stability is listed on the product label. The product should not be used beyond this date.

**Procedure**

<b>Procedure:</b>	<p>The product is manufactured in a ready to use form. It is intended to serve as a direct replacement to pre-existing materials distributed by the OEM. For a detailed description of the use of this reagent, refer to the Instrument's Operator Manual.</p> <ul style="list-style-type: none"><li>- High G/L requires no room temperature equilibration and may be used directly from the refrigerator.</li><li>- Do not use an ampule that is frozen. Allow it to thaw and mix thoroughly before use.</li><li>- Analyze the High G/L solution twice a week on your 800 series system<ol style="list-style-type: none"><li>1. Perform a successful 2-point calibration</li><li>2. Mix ampule contents by gently inverting several times, then tap liquid back into the ampule.</li><li>3. Carefully snap open the ampule</li><li>4. Analyze the High G/L solution. Sample the contents of the ampule using a syringe or an adapter. Refer to <i>Analyzing Samples</i>, in Section 3 of the 800 series Operator's Manual.<ul style="list-style-type: none"><li>▪ Replace the glucose sensor if results are below 800 mg/dL (44.4) mmol/L).</li><li>▪ Replace the lactate sensor if results are below 20 mmol/L (180 mg/dL).</li></ul></li></ol></li></ul>
-------------------	---

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