Mission CliNICAL™ Calibrator

DD-93100AD Calibrator
DD-93100BD Diluent
DD-93100 Mission CliNICAL™ Calibrator

2011/06

X90828

Calibrator
Diluent

X90829
R9F138

ENGLISH

INTENDED USE

Mission CliNICAL is a single level calibrator based on lyophilized human serum which contains the following analytes Acid Phosphatase (Total), Albumin, Alkaline Phosphatase (ALP), Amylase, Aspartate Aminotransferase (AST/GOT), Bicarbonate (CO₂), Bilirubin (Direct), Bilirubin (Total), Calcium, Chloride, Cholesterol (Total), Cholinesterase, Creatine Kinase (CK), Creatinine, Gamma Glutamyltransferase (GGT), Glucose, Iron, Lactate (Lactic acid), Lactate Dehydrogenase (LDH), Lipase, Lithium, Magnesium, Phosphorus, Potassium, Protein-Total, Sodium, Triglycerides, Urea, Urea Nitrogen, and Uric Acid. The concentrations and activities are suitable for calibration of clinical chemistry assays both manually and on a wide range of automatic analyzers.

PRODUCT DESCRIPTION

Mission CliNICAL™ is made up of lyophilized human serum and a diluent for reconstitution. The concentrations and activities of the analytes are suitable for calibration of clinical chemistry assays on a broad range of automatic analyzers.

PREPARATION FOR USE

Mission CliNICAL™ must only be reconstituted using the following procedure:

- Carefully open the vial so as to prevent any loss of calibrator material.
- Pipette exactly 3.0 mL of the diluent into the vial of lyophilized material.
- Replace the rubber stopper. Allow 30 minutes for reconstitution.
- Swirl gently several times during the reconstitution period to ensure contents are completely and well dissolved.
- Immediately prior to use, mix the contents by inverting the vial.

STORAGE AND STABILITY

Store the unreconstituted Calibrator at 2-8°C. The unreconstituted Calibrator is stable until the expiry date listed on the vial label at 2-8°C.

Once reconstituted with the diluent provided, the Mission CliNICAL™ is stable for 72 hours at 2-8°C and at least 20 days at -20°C when frozen once (see limitations).

LIMITATIONS

After reconstitution, Total Acid Phosphatase is stable for at least 1 day at 2-8°C and at least 20 days at -20°C when frozen once.

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WARNING

Each human donor unit used to manufacture this control was tested by FDA accepted methods and found non-reactive for Hepatitis B Surface Antigen (HBsAg), antibody to Hepatitis C (HCV) and antibody to HIV-1/HIV-2. This product may also contain other human source material for which there are no approved tests. In accordance with good laboratory practice, all human source should be considered potentially infectious and handled with the same precautions used with patient specimen. Material Safety Data Sheet (MSDS) available for professional users on www.diamonddiagnostics.com.

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ROCHE HITACHI CHEMISTRY ANALYZERS

ANALYTE	METHOD
Acid Phosphatase	Alpha-naphthylphosphate, Colorimetric
Albumin	(BCG) Bromcresol Green
ALP	PNPP, AMP Buffer, IFCC 2002
Amylase	CNPG3
AST (GOT)	IFCC 2002 (UV without P5P)
Bilirubin-Direct	Diazotization, DPD
Bilirubin-Total	Diazotization, DPD
Calcium-Total	Arsenazo III
Carbon Dioxide	PEP Carboxylase
Chloride	ISE Indirect
Cholesterol-Total	Cholesterol, esterase, oxidase, peroxidase
Cholinesterase	Butyrylthiocholin
CK (ATP)	NAC activated
Creatinine	Enzymatic
Glucose	Hexokinase
GGT	P-Nitrophenol
Iron	Ferene, FerroZine
Lactate	Colorimetric, Enzymatic
LDH	LDH-L (IFCC 2002)
Lipase	Colorimetric
Lithium	ISE Direct
Magnesium	Xylydyl Blue
Phosphorus	Ammonium/Phospho Molybdate
Potassium	ISE Indirect
Protein-Total	Biuret
Sodium	ISE indirect
Triglyceride	Enzymatic, GPO
Urea	Urease, UV / GLDH
Urea Nitrogen	Urease, UV / GLDH
Uric Acid	Uricase, colorimetric

CALIBRATION VALUES	
CONVENTIONAL UNITS	SI UNITS
29.5 U/L	0.493 µmol/L.sec
3.40 g/dL	34.3 g/L
246 U/L	4.09 µmol/L.sec
405 U/L	6.77 µmol/L.sec
105 U/L	1.76 µmol/L.sec
1.50 mg/dL	25.5 µmol/L
6.90 mg/dL	119 µmol/L
11.4 mg/dL	2.86 µmol/L
21.3 mEq/L	21.3 mmol/L
102 mEq/L	102 mmol/L
229 mg/dL	5.92 µmol/L
4911 U/L	82.0 µmol/L.sec
408 U/L	6.80 µmol/L.sec
3.70 mg/dL	330 µmol/L
248 mg/dL	13.8 µmol/L
149 U/L	2.49 µmol/L.sec
185 µg/dL	33.0 µmol/L
42.1 mg/dL	4.68 mmol/L
324 U/L	5.42 µmol/L.sec
169 U/L	2.83 µmol/L.sec
1.83 mEq/L	1.83 mmol/L
5.03 mg/dL	2.07 µmol/L
6.25 mg/dL	2.02 µmol/L
5.53 mEq/L	5.53 mmol/L
5.09 g/dL	50.9 g/L
158 mEq/L	158 mmol/L
201 mg/dL	2.27 µmol/L
104 mg/dL	17.3 µmol/L
48.6 mg/dL	17.3 µmol/L
8.67 mg/dL	516 µmol/L

* Instruments Include: Roche Hitachi 704, 717, 902, 904, 911, 912, 917, Modular

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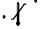
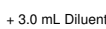
BECKMAN CHEMISTRY ANALYZERS

ANALYTE	METHOD
Albumin	(BCG) Bromocresol Green
ALP	PNPP, AMP Buffer, IFCC 2002
Amylase	CNPG3
AST (GOT)	IFCC 2002 (UV without P5P)
Bilirubin-Direct	Diazotization, DPD
Bilirubin-Total	Diazotization, DPD
Calcium-Total	Arsenazo III
Chloride	ISE Indirect
Cholesterol-Total	Cholesterol, esterase, oxidase, peroxidase
CK (ATP)	NAC activated
Creatinine	Enzymatic
Glucose	Hexokinase
GGT	P-Nitrophenol
Iron	Ferene, FerroZine
Lactate	Colorimetric, Enzymatic
LDH	LDH-L (IFCC 2002)
Lipase	Colorimetric
Lithium	ISE Direct
Magnesium	Xylydyl Blue
Phosphorus	Ammonium/Phospho Molybdate
Potassium	ISE Indirect
Protein-Total	Biuret
Triglyceride	Enzymatic, GPO
Urea	Urease, UV / GLDH
Urea Nitrogen	Urease, UV / GLDH
Uric Acid	Uricase, colorimetric

CALIBRATION VALUES	
CONVENTIONAL UNITS	SI UNITS
2.93 g/dL	29.3 g/L
249 U/L	4.13 µmol/L.sec
312 U/L	5.22 µmol/L.sec
88.8 U/L	1.48 µmol/L.sec
0.960 mg/dL	16.4 µmol/L
9.86 mg/dL	169 µmol/L
10.6 mg/dL	2.66 µmol/L
114 mEq/L	114 mmol/L
217 mg/dL	5.61 µmol/L
377 U/L	6.30 µmol/L.sec
3.00 mg/dL	264 µmol/L
232 mg/dL	12.9 µmol/L
129 U/L	2.16 µmol/L.sec
96.3 µg/dL	17.2 µmol/L
46.1 mg/dL	5.13 mmol/L
268 U/L	4.47 µmol/L.sec
135 U/L	2.25 µmol/L.sec
1.83 mEq/L	1.83 mmol/L
3.84 mg/dL	1.58 µmol/L
5.68 mg/dL	1.84 µmol/L
5.54 mEq/L	5.54 mmol/L
4.38 g/dL	43.8 g/L
165 mg/dL	1.86 µmol/L
105 mg/dL	37.3 µmol/L
49.1 mg/dL	17.5 µmol/L
8.79 mg/dL	522 µmol/L

* Instruments Include: Beckman CX3, CX4, CX5, CX7
 Beckman CX3 Delta, CX4 Delta, CX5 Delta, CX7 Delta, CX9 ALX
 Beckman CX3 PRO, CX4 PRO, CX5 PRO, CX7 PRO, CX9 PRO

 For In Vitro Diagnostic Use In Vitro Diagnostic Usage In Vitro Para Uso Diagnostico In Vitro Utilizar Apenas Em Diagnostico In Vitro Ti In Vitro diagnostic 体外诊断使用 Для использования в диагностике In Vitro	 European Conformity CE-Konformitätskennzeichnung Conformité aux normes européennes Conformidad europea Conformidade com as normas europeias Europaisk overensstemmelse 符合欧 Европейская Адекватность	 Temperature Limit Temperaturlimit Limite de température Limite de temperatura Limite de temperatura Temperaturgrenze 温度范围限制 Температурные ограничения	 Consult Instructions for Use Gebrauchsanweisung beachten Consulter la notice d'emploi Consulte las instrucciones de uso Consulte as instruções de utilização Benyt bruksanvisningen 参考说明书使用 Рекомендации по применению	 Lot Number Chargen-Nr. Número de lot Número de lote Número de lote Batchnummer 批号 Номер серии	 Use by (YYYY-MM-DD) Verwendbar bis (JJJJ-MM-TT) Date de péremption (AAAA-MM-JJ) Usar hasta el (AAAA-MM-DD) Utilizar até (AAAA-MM-DD) Avend for (AAAA-MM-DD) 有效期至(YYYY-MM-DD) Используется дата (год-месяц-день рождения)	 Caution Vorsicht Attention Precaución Cuidado Forsigtig 注意 Меры предосторожности	 Manufactured by Hergestellt von Fabriqué par Fabricado por Fabricado por Fremstillet af ***制造 оказатель	 Authorized Representative Bevollmächtigter Représentant agréé Representante autorizado Representante autorizado Autoriseret representant 授权的代表 Санкционированный представитель	 + 3.0 mL Diluent Reconstitute With Rekonstituieren mit À reconstituer avec Reconstituir con Reconstituier com Rekonstituere med 用***稀释 Восстанавливать	 Catalog Number Katalognummer Número de catalogue Número de catálogo Número de catálogo Katalognr. 产品编号 Номер каталога
---	---	---	--	---	--	--	---	--	---	---