**Product And Manufacturer Information**

- Contents:** 1 K+ Electrode non-refillable.
- Intended Use:** Use with Diamond SMARTLYTE, CARELYTE, GEMLYTE, Roche AVL 91xx, 98x, Idexx VetLyte, Alfa-Wassermann StarLyte III Electrolyte Analyzers.
- Storage:** Store the K+ Electrode at room temperature (18-25°C) until you are ready to use it in your system.
- Precautions:** Please check that the electrode body is  $\frac{3}{4}$  filled with Fill Solution prior to installation and no air bubbles in fill chamber near membrane sample path area.

Diamond Diagnostics, 333 Fiske Street, Holliston, MA.  
**For Technical Assistance call:**  
Diamond Diagnostics Technical Services at 1-508-429-0450

**Installation**

- 1 Check that the electrode is  $\frac{3}{4}$  filled with Fill Solution.
- 2 Check for air bubbles in the fill chamber near sample path. If there are any, tap the electrode body to dislodge air bubbles.
- 3 Verify that the O-rings are in place on the electrode body.
- 4 Install electrode in your system.
- 5 Allow the electrode to equilibrate for at least 15 minutes.
- 6 Perform a two-point calibration.
- 7 Analyze a quality control sample.

**Warranty**

Warranty is calculated 6 months from the date of shipment from Holliston, MA.

**Product And Manufacturer Information**

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Diamond Diagnostics Technical Services at 1-508-429-0450

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