Product And Manufacturer Information

- Contents:** 1 – Reference Electrode.
- Intended Use:** Use with Cobas ISE Modules.
- Storage:** Store the reference electrode at room temperature (18-25°C) until you are ready to use it in your system.
- Precautions:** The reference electrode is designed to be shipped and stored dry.

Diamond Diagnostics
333 Fiske Street, Holliston, MA.

For Technical Assistance call:
Diamond Diagnostics Technical Services at 1-508-429-0450

Installation and Storage

Installation:

- 1 Install electrode in your system.
- 2 Perform a two-point calibration
- 3 Allow the Reference electrode to equilibrate for at least 30 minutes.
- 4 Analyze a quality control sample.

Removing the electrode for Storage:

Note: Do not allow the reference solution to dry in the electrode sample path or electrode tubing as this affects electrode performance.

- 1 Store the Reference electrode dry if you are shutting the system down for an extended period of time. There is no need to remove the electrode if the system is in the standby mode.
- 2 Complete the following procedure if you want to remove and store the Reference electrode:
 - a Remove electrode form system
 - b Rinse the tubing and sample path alternately with de-ionized water and air several times.
- 3 Store the dry Reference electrode at room temperature (18-25°C) until you are ready to use it.

Warranty

Warranty is calculated 6 months from the date of shipment from Holliston, MA.

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