Manufacturer and Product Information

Diamond Diagnostics, 333 Fiske Street, Holliston, MA.

For Technical Assistance call:

Diamond Diagnostics Technical Services at 1-508-429-0450

Intended Use: Diamond Slope tHB is used to provide a tHB calibration point for Bayer/Ciba-Corning 270 and 800 Instruments.

Summary and Principle: This product is intended to serve as a functional equivalent to pre-existing material distributed by the Original Equipment Manufacturer (OEM).

Reagents: CD-473120D Slope tHB - 10 x 1.8 ml
Containing amaranth and a viscosity adjuster

Value Assignment: Each reagent is tested for each calibrating analyte. tHB reference is made on the OEM Analyzer.

tHB Values for:

LOT R6I137 – EXP 08/09	
Ciba-Corning 270 – 14.2 g/dL	Expected Range – 14.0 – 14.6 g/dL
Ciba-Corning 800 – 13.6 g/dL	Expected Range – 13.5 – 13.7 g/dL

For in vitro diagnostics use.

Cautions: Exercise normal laboratory precautions. If contact occurs with skin, rinse affected area with water. If contact with eyes occurs, immediately rinse with copious amount of clean water or eye rinse. In cases of accidental ingestion, contact a physician immediately. Dye can be removed from clothes and skin by washing with warm soap and water.

Stability: Package stability is listed on the product label. The product should not be used beyond this date. Store upright at room temperature, 18-25°C and avoid exposure to direct light.

Procedure

Procedure: The product is manufactured in a ready to use form. It is intended to serve as a direct replacement to pre-existing materials distributed by the OEM. For a detailed description of the use of this reagent, refer to the Instrument's Operator Manual.

Quality Control: Diamond Diagnostics suggests the use of commercially available control material with results assayed for the instrument used. Controls should be run at Normal and Abnormal levels. Diamond Diagnostics suggests measuring controls before patient samples are run and following instrument maintenance.

Limitations

LIMITATIONS: If the instrument fails calibration or controls do not measure within acceptable range when Diamond Diagnostics products are used, Diamond Diagnostics suggests the following:

1. Verify that the reagents and internal calibrators used to standardize the instrument are correct for the instrument, have adequate expiration, and do not contain visually evident contamination.
2. Follow the procedures delineated within the Operator's Manual listed under Troubleshooting.
3. Ensure that all appropriate Maintenance Procedures, as listed in the Operator's Manual, have been performed.
4. If problems still exist, contact Diamond Diagnostics' Technical Service Department.

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LOT R61137 – EXP 08/09	
Ciba-Corning 270 – 14.2 g/dL	Expected Range – 14.0 – 14.6 g/dL
Ciba-Corning 800 – 13.6 g/dL	Expected Range – 13.5 – 13.7 g/dL

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