Manufacturer and Product Information

Diamond Diagnostics, 333 Fiske Street, Holliston, MA.

For Technical Assistance call:

Diamond Diagnostics Technical Services at 1-508-429-0450

Intended Use: Wash concentrate is intended for use with Beckman Synchron CX Systems. The wash solution is formulated as a reagent diluent and as a cleaning solution for the same probe.

Short History of Methodology: The reconstituted wash concentrate is a dilute solution of wetting agents and preservatives. It is specially formulated to work as a reagent diluent and wash solution for the Synchron CX 3 and its chemistries. This allows the reagents to be used as a concentrate thereby dramatically improving their open stability.

Contents: Each Wash Solution Kit contains the following:
1 – Wash Concentrate: Six bottles each containing a 250mL solution of alcohol, surfactants, and stabilizers necessary for optimum system operation.
2 – One overlay label.

For *in vitro* diagnostic use only

Cautions: Wash concentrate is a flammable alcohol solution. Do not work with this reagent in the presence of an open flame or a hot heating element. Keep container closed.

Stability & Storage: Store wash concentrate at room temperature. Wash concentrate is stable to the expiration date listed on the bottle. The working solution is stable for thirty days after reconstruction.

Wash Solution Preparation

Procedure: The working solution is prepared by diluting 250 mL of the wash concentrate to 10 liters with deionized water. Before using this freshly reconstituted wash solution, inspect in-line filters and replace as necessary. Always discard unused wash solution thirty days after reconstitution.

Each kit supplies an overlay label. Apply overlay label to the ten-liter bottle and record the date prepared.

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