Manufacturer and Product Information

Diamond Diagnostics, 333 Fiske Street, Holliston, MA.

For Technical Assistance call:

Diamond Diagnostics Technical Services at 1-508-429-0450

Contents:

- 1 – pH Electrode
- 1 – Cleaning Solution (RD-943905E)
- 1 – Cuvette

Intended Use:

Use with Radiometer ABL 5xx Series analyzers.

Storage:

Store the pH electrode at room temperature (18-25°C) until you are ready to use it in your system.

Precautions:

Following pH electrode replacement, automatic calibrations are changed to the following sequence:

1. First 4 hours, alternate in 30 minute intervals Cal 1 and Cal 2, ending in Cal 2.
2. Next 20 hours, every hour Cal 1, every fourth hour alternate with Cal 2.

Procedure

Installation:

1. Unpack the new pH electrode. Preparation requires 25 hours.
2. Add 10 drops of Electrode cleaner, RD-943905E, into the Cuvette provided with the electrode.
3. Soak the electrode in the Cuvette for 1 hour.
4. Remove the electrode and rinse with plenty of distilled or deionized water.
5. Empty the Cuvette and rinse with distilled or deionized water. Fill with distilled/deionized water up to one third the height of the Cuvette.
6. Soak the electrode in distilled/deionized water for 24 hours. The electrode may drift excessively if soaked for a shorter time.
7. Remove the cover of the temperature controlled electrode unit.
8. Press the pH amplifier housing down, pull the metal tab outward, and lift the amplifier housing.
9. Remove the old pH electrode.
10. Remove the newly prepared electrode from the Cuvette with water. Gently shake excess water from the electrode.
11. Using a dry clean cloth or paper towel, thoroughly wipe and dry the gold contacts of the electrode.
12. Place the newly prepared electrode in the correct section of the measuring chamber.
13. Gently press the amplifier housing down onto the electrode.
14. Record in the maintenance schedule that the pH electrode has been replaced.
15. If all maintenance steps have been performed, place the cover over the temperature controlled electrode unit and press **Restart Analyzer** key below the display in scheduled maintenance menu.

Warranty

Warranty = 12 months: Warranty is calculated from date of shipment from Holliston, MA.

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