Product And Manufacturer Information

Contents:	1 - Li ⁺ Electrode non-refillable.
Intended Use:	Use with Diamond SMARTLYTE, CARELYTE, GEMLYTE, Roche AVL 91xx, 98x, Idexx VetLyte, Alfa-Wassermann StarLyte III Electrolyte Analyzers.
Storage:	Store the Li ⁺ electrode at room temperature (18-25°C) until you are ready to use it in your system.
Precautions:	Please check that the electrode is $\frac{3}{4}$ filled with Fill Solution prior to installation and no air bubbles in the fill chamber near the membrane sample path area.

Diamond Diagnostics, 333 Fiske Street, Holliston, MA.
For Technical Assistance call:
Diamond Diagnostics Technical Services at 1-508-429-0450

Installation

- 1 Check that the electrode is $\frac{3}{4}$ filled with Fill Solution.
- 2 Check for air bubbles in the fill chamber near sample path. If there are any, tap the electrode body to dislodge air bubbles.
- 3 Verify that the O-rings are in place on the electrode body.
- 4 Install electrode in your system.
- 5 Allow the electrode to equilibrate for at least 15 minutes.
- 6 Perform a calibration.
- 7 Analyze quality control samples.

Warranty

Warranty is calculated 3 months from the date of shipment from Holliston, MA.

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Diamond Diagnostics Technical Services at 1-508-429-0450

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